PROVOX[®] LaryTube[™]

Sizer Kit


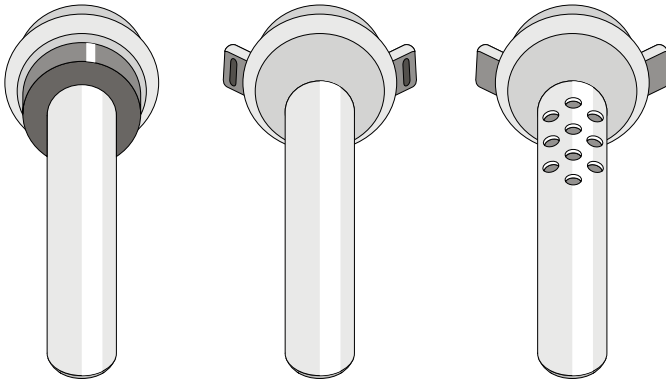
Figure 1.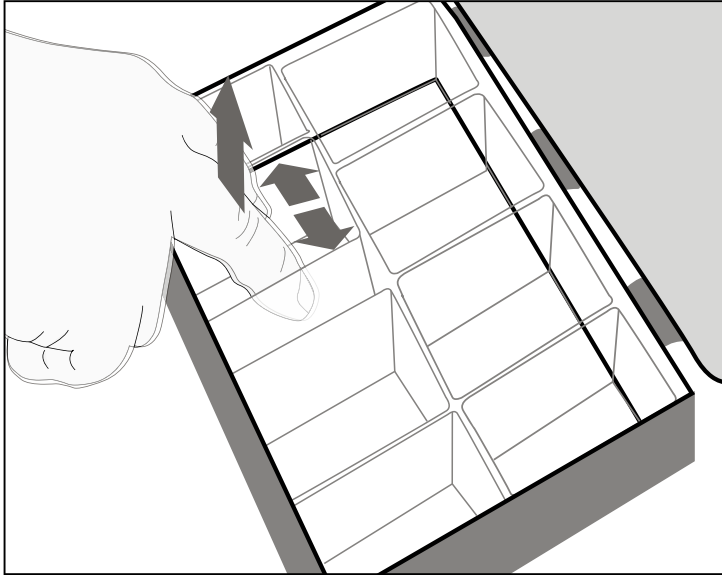

Prescription information

CAUTION: United States Federal law restricts this device to sale, distribution and use by or on order of a physician or a licensed practitioner. The availability of this product without prescription outside the United States may vary from country to country.

Disclaimer

Atos Medical offers no warranty - neither expressed nor implied - to the purchaser hereunder as to the lifetime of the product delivered, which may vary with individual use and biological conditions. Furthermore, Atos Medical offers no warranty of merchantability or fitness of the product for any particular purpose.

Patents and trademarks

Provox® is a registered trademark owned by Atos Medical AB, Sweden. Provox® LaryTube™ is a trademark by Atos Medical AB.

For information about protective rights (e.g. patents), please refer to our web page www.atosmedical.com/patents.

Manufacturer; Hersteller; Fabrikant; Fabricant; Produttore; Fabricante; Fabricante; Tillverkare; Producent; Produzent; Valmistaja; Framleiðandi; Tootja; Ražotājs; Gamintojas; Výrobce; Gyártó; Výrobca; Proizvajalec; Producent; Producător; Proizvođač; Proizvođač; Κατασκευαστής; Производител; Üretici; მწარმოებელი; Ἀρτοποιός; Įstehsalci; Производитель; Produšen; Pengilang; 製造業者; 제조사; 製造商; 製造商; היצרן; الجهة المصنعة

Date of manufacture; Herstellungsdatum; Productiedatum; Date de fabrication; Data di fabbricazione; Fecha de fabricación; Data de fabrico; Tillverkningsdatum; Fremstillingsdato; Produksjonsdato; Valmistuspäivä; Framleiðsludagsetning; Valmistamise kuupäev; Ražošanas datums; Pagaminimo data; Datum výroby; Gyártás időpontja; Dátum výroby; Datum proizvodnje; Data produkcji; Data fabricației; Datum proizvodnje; Datum proizvodnje; Ημερομηνία κατασκευής; Дата на производство; Üretim tarihi; წარმოების თარიღი; Ἀρτοποιησιμὸν ἀναμνηστικὸν; Įstehsal tarixi; Дата изготовления; Tanggal produksi; Tarikh pembuatan; 製造日; 제조일자; 製造日期; 製造日期; ייצור, תאריך ייצור; تاريخ التصنيع

Use-by date; Verfallsdatum; Houdbaarheidsdatum; Date limite d'utilisation; Data di scadenza; Fecha de caducidad; Data de validade; Används senast; Anvendes inden; Brukes innen; Viimeinen käyttöpäivä; Sijasti notkunardagur; Kõlblik kuni; Derīguma termiņš; Naudoti iki; Datum expirace; Felhasználhatóság dátuma; Dátum najneskoršej spotreby; Rok uporabnosti; Data ważności; Data expirării; Rok uporabe; Rok upotrebe; Ημερομηνία λήξης; Срок на годност; Son kullanim tarihi; ვარგისობა; Ἀποχρηστικὸν ἡμερομηνίαν; Son istifadə tarixi; Срок годности; Gunakan sebelum tanggal; Tarikh guna sebelum; 使用期限; 사용 기한; 使用截止日期; 使用截止日期; תאריך אחרון לשימוש; تاريخ انتهاء الصلاحية

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Intended use

The Provox LaryTube Sizer Kit is intended for use by the prescribing clinician to determine the size(s) of LaryTube that should be prescribed to the laryngectomized patient.

The Sizer Kit should be used only by a prescribing clinician who has read the LaryTube Manual. A copy of that manual comes with the Sizer Kit. It can also be viewed on the Internet at www.atosmedical.com.

The Sizer LaryTubes are intended for the sizing procedure only. After the correct size(s) have been determined, a new LaryTube(s) shall be prescribed to the patient for actual use. LaryTube order information is found on the last page of this manual.

Contraindications

The Sizer Kit in itself does not have specific contraindications.

The Provox LaryTubes contained in the LaryTube Sizer Kit are not intended for patients requiring mechanical ventilation.

Device description

The Sizer Kit is a box which contains samples (“Sizers”) of a variety of commercially available Provox LaryTubes. The sizes of these Sizers and actual Provox LaryTubes are the same. The size is indicated on the products and both diameter and length are indicated on the chart inside the box.

Each Sizer in the Sizer Kit is stored in an individual removable polypropylene box. This makes it possible for the prescribing specialist to remove the individual storage boxes with the Sizers from the outer storage box individually. This allows for hygienic handling of both the Sizers and the storage boxes.

After each sizing session, the Sizer(s) with its individual storage box(es) must be cleaned, disinfected, dried and steam sterilized according to the accompanying “Instructions for cleaning and sterilization”. The outer storage box must also be cleaned if contaminated. The Sizer LaryTubes and their individual removable storage boxes are thereafter put back at the appropriate position as indicated in the bottom of the outer storage box.

WARNINGS

- TO AVOID THE TRANSFER OF DISEASE, NEVER USE A CONTAMINATED SIZER LARYTUBE.

After using a Sizer LaryTube, it becomes contaminated through handling and contact with the patient’s stoma. To avoid the transfer of disease from one patient to another, use appropriate hygienic procedures and do not put the LaryTube and its individual storage box back in the outer storage box after use. Make sure that each used Sizer and individual storage box are cleaned, disinfected, dried and steam sterilized. The outer storage box must also be cleaned if contaminated.

- The Sizer LaryTubes are intended for the sizing procedure only. After the correct size(s) have been determined, new LaryTube(s) shall be used by the patient.

PRECAUTIONS

- To reduce risks of causing damage to the tracheostoma during the Sizing procedure:
 - Thoroughly read the LaryTube Manual before using the LaryTube Sizer Kit in order to understand the Warnings, Precautions and Instructions for use of the LaryTube during insertion and replacement.
 - Insert and remove the Provox LaryTube with caution, never use force.
 - Only use water-soluble lubricants. Oil-based lubricants (e.g., Vaseline) may weaken, damage or destroy the product and shall not be used.
- The Sizers and the individual storage boxes are designed to withstand 10 cleaning and steam sterilization procedures at 137°C/278°F for 3 minutes or at 134°C/273°F for 18 minutes. Always ensure mechanical integrity before each use.
- The outer storage box is not designed to withstand any steam sterilization.
- Make sure that no silicone residues are left in the holes or the lumen of the LaryTube after fenestration.
- If the patient is using a voice prosthesis, be careful during insertion of the LaryTube. You may accidentally dislodge the voice prosthesis, which could lead to aspiration of the prosthesis.

Adverse events

The following are possible risks associated with the use of LaryTube Sizer Kit:

- Transfer of disease(s). In order to reduce the risk, use appropriate hygienic procedures. Clean, disinfect, dry and steam sterilize the Sizer LaryTube and individual storage box.
- Use of wrong size Sizer LaryTube during sizing procedure. To reduce the risk, make sure that the correct Sizer LaryTube is packaged in its correct position as indicated in the bottom of the outer storage box.

Instructions for use

Checkout procedures

Do not use the LaryTube Sizer Kit if the package is damaged or the tamper seal has been opened.

Check that correct size(s), as listed with the product, are available in the Sizer Kit.

Make sure that the Sizer Kit has been cleaned and sterilized as appropriate.

Always ensure mechanical integrity of the LaryTube before each use.

Removing the Sizer from the outer storage box

First remove the individual storage box with the Sizer from the outer storage box without contaminating the remaining LaryTubes, individual storage boxes, and outer storage box (Figure 1). Then remove the Sizer from its individual storage box.

Choosing the right Provox LaryTube

There is a wide variety in stoma shapes and diameters that are suitable for LaryTube use.

In any case, the proper fit may result from trying out several sizes.

Diameter

The goal is to create a comfortable and airtight fit between the Provox LaryTube and the tracheostoma. Measure the size of the tracheostoma (largest diameter) with a ruler and fit the Provox LaryTube accordingly.

Length

By trying different lengths of the Sizers, the most suitable length for the individual patient can be chosen.

Different types of LaryTubes

The LaryTube is available in three versions:

- The Fenestrated version, designed for use in combination with a voice prosthesis. Several small fenestrations are preferred over a single large one to avoid locking and accidental dislocation of the voice prosthesis.
- The Standard version, made for use without a voice prosthesis or when customized fenestration is preferred.
- The LaryTube with Ring version, to be worn with an adhesive base plate. This is recommended if a cannula is only used during the night, or if attachment by means of an adhesive is preferred over other attachments such as a Provox TubeHolder or Provox LaryClip.

Fenestration

The LaryTube comes both pre-fenestrated (the Fenestrated version), or can be fenestrated individually (the Standard version or the version with Ring, see LaryTube manual).

If the patient uses a voice prosthesis or shunt phonation, the LaryTube can be fenestrated for optimal airflow and functionality for tracheoesophageal voicing. Please see the LaryTube Manual for instructions on how to fenestrate a LaryTube.

Cleaning and sterilization

For the Sizer LaryTubes and the individual storage boxes: Clean, disinfect and steam sterilize the Sizers and their individual storage boxes separately. Put the individual storage box upside down during cleaning in washer-disinfector and during sterilization. Read the “Instructions for cleaning and sterilization”. If this information is missing, please contact your supplier or visit our website on www.atosmedical.com.

The outer storage box has been tested to withstand the alkaline detergent; Suma Med Super LpH® (Johnson Diversey) and the disinfectant; Gigasept® Instru AF (Schülke&Mayr). Cleaning and disinfection of the separated outer storage box and size indicating sheet can be performed with this detergent and disinfectant according to the methods described in the “Instructions for cleaning and sterilization”.

Repacking after cleaning and sterilization

Make sure that the outer storage box is clean and dry before putting the reprocessed individual storage boxes and Sizers back in the outer storage box.

Check the Sizers for mechanical integrity before repacking.

Put the Sizers in their individual storage boxes in the correct location according to the size indicator in the bottom of the outer storage box.

Make sure to use appropriate hygienic procedures to avoid contamination of the Sizers and individual storage boxes.

How supplied

The device is manufactured in a cleanroom and supplied non-sterile.

Device lifetime

The Sizer LaryTubes shall be replaced every 2 years, or after 10 cleaning and steam sterilization procedures, whichever comes first. The individual storage boxes withstand 10 cleaning and steam sterilization procedures and cannot be replaced separately. Check mechanical integrity of the device prior to each use.

Disposal

Always follow medical practice and national requirements regarding biohazards when disposing of a used medical device.

Ordering information

See last page of this manual.

Date of printing

See version number on the back cover of this manual.

User assistance information

For additional help or information please see back cover of this manual for contact information.

Reporting

Please note that any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the national authority of the country in which the user and/or patient resides.

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