PROVOX[®] Luna[®]

ShowerAid

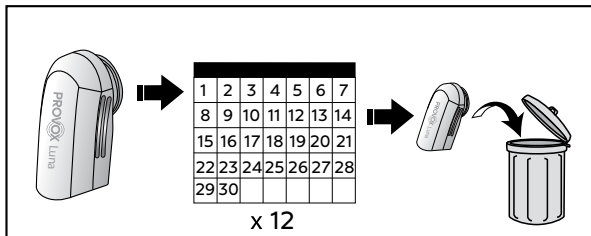
Fig. 1

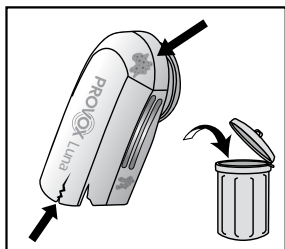
Fig. 2

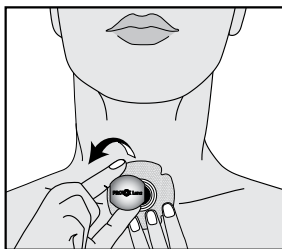
Fig. 3

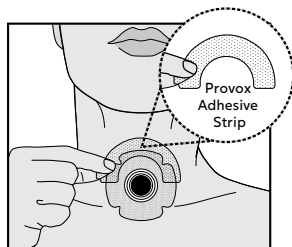
Fig. 4

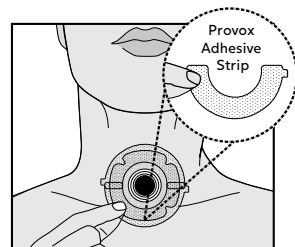
Fig. 5

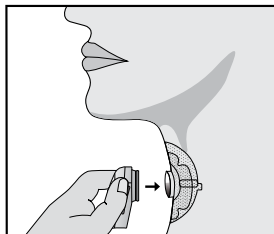
Fig. 6

Fig. 7

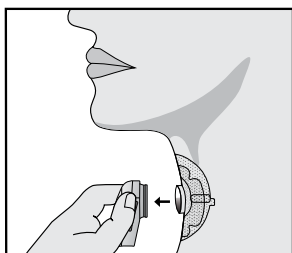
Fig. 8

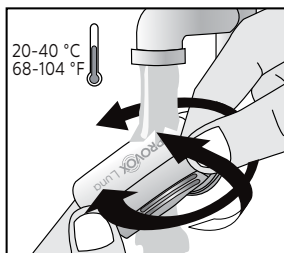
Fig. 9

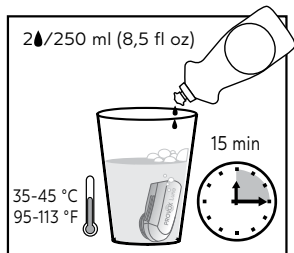
Fig. 10

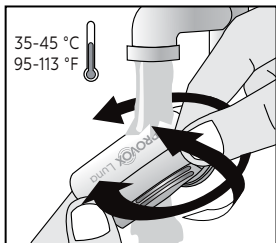
Fig. 11

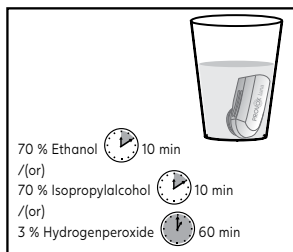
Fig. 12

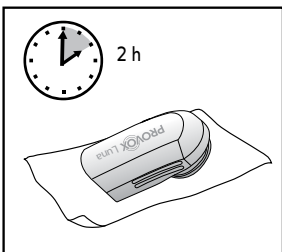
Fig. 13

Disclaimer

Atos Medical offers no warranty – neither expressed nor implied – to the purchaser hereunder as to the lifetime of the product delivered, which may vary with individual use and biological conditions. Furthermore, Atos Medical offers no warranty of merchantability or fitness of the product for any particular purpose.

Compatible Products; Kompatible Produkte; Compatibele producten; Produits compatibles; Prodotti compatibili; Productos compatibles; Produtos compatíveis; Kompatibla produkter; Kompatible produkter; Kompatible produkter; Yhteensopivat tuotteet; Samhæfar vörur; Ûhilduvad tooted; Suderinami gaminiai; Kompatibilní produkty; Kompatibilis termékek; Kompatibilné produkty; Združljivi izdelki; Zgodne produkty; Prognose compatibile; Kompatibilni proizvodi; Kompatibilni proizvodi; Συμβατά προϊόντα; Совместимые изделия; Uyumlu Ürünler; Համատեղելի արտադրանքներ; Produk yang Serasi; 互換性のある製品; 호환 가능한 제품; 相容產品; מוצרים תואמים; المنتجات المتوافقة

Provox Luna Adhesive
Provox Luna Set

Contents

EN - ENGLISH	7
DE - DEUTSCH	9
NL - NEDERLANDS	11
FR - FRANÇAIS	13
IT - ITALIANO	15
ES - ESPAÑOL	17
PT - PORTUGUÊS	19
SV - SVENSKA	21
DA - DANSK	23
NO - NORSK	25
FI -SUOMI	27
IS - ÍSLENSKA	29
ET - EESTI	31
LT - LIETUVIŲ KALBA	33
CS - ČESKY	35
HU - MAGYAR	37
SL - SLOVENŠČINA	39
SK - SLOVENČINA	41
PL - POLSKI	43
RO - ROMÂNĂ	45
HR - HRVATSKI	47
SR - SRPSKI	49
EL - ΕΛΛΗΝΙΚΑ	51
TR - TÜRKÇE	53
HY - ՀԱՅԵՐԵՆԻՍԻ	55
RU - РУССКИЙ	57
MS - BAHASA MELAYU	59
JA - 日本語	61
KO - 한국어	63
ZHTW - 繁體中文	65
ZHCN - 简体中文	67
HE - עברית	70
AR - العربية	72

1. Intended Use

Provox Luna ShowerAid is used with the Provox Luna Adhesive while taking a shower to avoid water from entering the stoma. Single patient use.

2. Contraindications

None.

3. Warnings

Single patient use only! Reuse between patients may cause cross contamination.

4. Instructions for Use

4.1 Operating instructions

Remove Provox Luna HME from Provox Luna Adhesive by holding the adhesive (Fig. 3). Make sure that the adhesive is securely attached to the skin using Provox Adhesive Strip (Fig. 4-5). Attach Provox Luna ShowerAid before showering (Fig. 6). To remove, hold the device and pull sideways until the device disconnects (Fig. 8).

4.2 Cleaning and disinfection

Clean and disinfect Provox Luna ShowerAid before each use.

Only use drinking water to clean and rinse the device.

1. Rinse with tap water 20 – 40 °C in 1 minute on all sides and through the device (Fig. 9).
2. Clean with 2 drops of dish soap in 250 ml water 35 - 45 °C for 15 minutes (Fig. 10).
3. Rinse with tap water 35 – 45 °C in 15 seconds on all sides and through the device (Fig. 11).
4. Disinfect (Fig. 12) with one of the following methods:
 - a. 70 % Ethanol for 10 minutes
 - b. 70 % Isopropylalcohol for 10 minutes
 - c. 3 % Hydrogenperoxide for 60 minutes.
5. Let dry in air for 2 hours (Fig. 13).

5. Storage instructions

Store Provox Luna ShowerAid in a clean and dry container at room temperature. Protect from direct sunlight.

6. Device lifetime and disposal

Provox Luna ShowerAid may be used for a maximum period of 12 months (Fig. 1). If the device shows any signs of damage, it shall be replaced earlier (Fig. 2).

Always follow medical practice and national requirements regarding biohazard when disposing of a used medical device.

7. Reporting

Please note that any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the national authority of the country in which the user and/or patients resides.

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© Coloplast A/S. All rights reserved.
Atos and the Atos Medical logo are trademarks of Coloplast A/S.
Atos Medical UK Ltd Tottle Road
Atos Medical AB